



Adhesive Skin Collection Kit Instructions



Troubleshooting QNS (Quantity Not Sufficient)? Try This!


Use a sanitized, blunt firm instrument (i.e the bottom of the sharpie provided) to rub the Smart Sticker onto the lesion and repeat for each sticker to enhance pressure.



Reminder: **No thick hair or blood** on samples

Ensure all required information is filled out and accurate on the Sample Collector Trifold. **This information is required for the sample to be processed.**

For help scheduling a pickup, call customer service at 866-450-4223 ext 156.

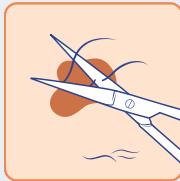

STEP 1:

Trim excessive hair with scissors. Ensure no blood.



STEP 2:

Fill out front of Sample Collector Trifold. **All information is required for sample to be processed.**




STEP 3:

Thoroughly cleanse lesion site with enclosed 70% alcohol pad.




STEP 4:

Thoroughly dry by gently pressing enclosed gauze pad over lesion. **Do not proceed to sample collection until lesion is completely dry.**

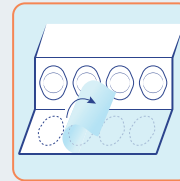


STEP 5:

Open Sample Collector Trifold. Remove and discard blue paper liner and "Remove Before Use" seal.

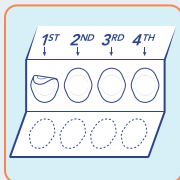


STEP 6:

Remove first Smart Sticker™

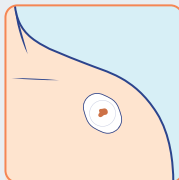


STEP 7:

Hold skin tight and center first Smart Sticker directly over lesion.

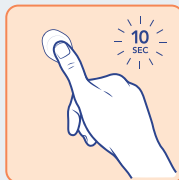


STEP 8:

Use thumb to **firmly** press Smart Sticker for 10 seconds. This will help maximize the amount of tissue collected.

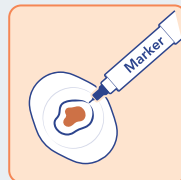


STEP 9:

Use the provided marker to **fully outline and enclose the lesion with minimal margin.**

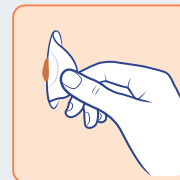


STEP 10:

Remove the Smart Sticker by slowly and continuously peeling it from the skin in one direction.

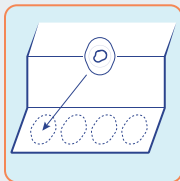


STEP 11:

Place the collected sample with adhesive side down directly on pre-marked oval area in Sample Collector Trifold.

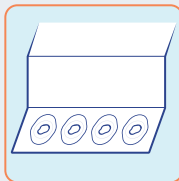


STEP 12:

Repeat steps 7-11 until all four (4) Smart Stickers have been used on the lesion.

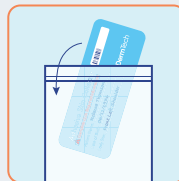

STEP 13:

Re-confirm patient information is completed and accurate on Sample Collector Trifold. Place sample into re-sealable foil bag containing desiccant pouch.

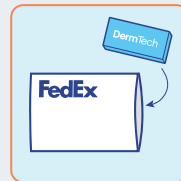

STEP 14:

Return sample in envelope provided within 72 hours of sample collection along with completed Test Requisition Form.

PLEASE NOTE:

- Do not freeze or refrigerate collected sample.
- Store at room temperature until shipment.
- Test results will typically be returned within 5 business days from receiving the sample.

DermTech